

FACE MASK REFI 96

Washable hypoallergenic face mask with filter refills and anti-fog nasal bar



Size 1
≤ 115mm

Size 2
> 115mm

The set protects you up to 420 uses



French made



Washable face mask and filters



Hypoallergenic TPE face mask



Recyclable face mask and filters



The face mask is approved by **LNE**

The filters are approved by the **DGA**

Blister face mask content



1 face mask + 1 strap + 1 filter support + **2 sachets of 7 filters** + 1 user manual

Filter blister content




4 sachets of 7 filters + 1 user manual

Safety / Approval / Warning

This face mask is approved for the general public (LNE report P205732). NT 1391 filters have a filtration efficiency of 95% (97% after 10 washes) of particles > 3µm in accordance with the 90% minimum required. The breathability of the face mask was measured at 658 L.m-2.s-1 in accordance with the required 96 L.m-2.s-1 minimum (DGA report RP / 20-4211). Despite all the care taken in the manufacture of these filters, it is the sole responsibility of the user, and in no case the manufacturer or supplier, to ensure the suitability of the filters with the characteristics of the working environment. The recommendations and methods of use should be followed scrupulously. These face masks are neither PPE according to regulation (EU) N ° 2016/425, nor medical devices according to directive 93/42 / CE.

CAUTION: if you are ill, this face mask is not suitable. Ask for your doctor's opinion. This mask is a supplement to barrier gestures (regular hand washing, physical distancing, reduction of contact with other people).

It is recommended to wear the mask on bare skin, avoiding contact with the hair (excluding the strap). Avoid touching and moving it. Never put it in the waiting position on the forehead or on the chin.

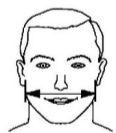


Bigonial distance

How to choose the size of my face mask

In order to best adapt to your morphology, choose the size of the face mask according to your bigonial distance :

Size 1 for faces ≤ 115 mm

Size 2 for faces > 115 mm

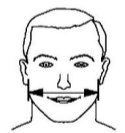

Bigonial distance

How to maintain the face mask and filters

Before opening : Store the set in a dry place out of direct sunlight. Store it at room temperature and protect it from moisture.

Before first use : clean all parts (filters + face mask).

- The face mask can be cleaned either in the dishwasher or in a washing machine (60 degrees for 30 minutes minimum, without adding any product), or with a hydroalcoholic solution, or in boiling water (for 7 minutes) .
- The face mask must be cleaned after each use.
- The filters can be washed up to 10 times in a washing machine (60 degrees for 30 minutes minimum), preferably in a net. After 10 washes, the filter is no longer guaranteed. Filters should be changed every 4 hours.
- To ensure effective cleaning, wash your hands with soap and water or with a hydroalcoholic solution before handling.
- When drying your face mask and filters, make sure that they are placed on a clean and disinfected surface. They must be steam ironed.
- When the face mask and filters are dry, store them in clean and airtight packaging (plastic bag, etc.).

How to assemble my face mask



Before touching the different parts, wash your hands with soap and water or a hydroalcoholic solution. Inspect each part. Make sure there are no abnormal holes, tears, or damage.

- Fit the strap on the face mask and pre-adjust it.
- Insert the filter into its support.
- Clip the filter support onto the face mask then insert the "T" shaped tabs in the location provided for this purpose to secure the hold of the filter support.

How to put on my face mask



- Position the face mask so as to cover the nose, mouth and chin.
- Pass the strap behind your head and adjust the settings if necessary.
- Then adjust the nose bar.
- You can cut the strap to make it easier to hold on some faces (optional). Possibility of using only a half strap.

For more information, you can contact us on www.acm-groupe.com Mail : contact@acm-groupe.com

Tél : 02 33 30 68 80

Address : SNACM, 15 route de BELLOU, 61600 LA SAUVAGERE (FRANCE)

